

Dr.Brett's Pets

User Manual for Your New Loupes

Introduction:

Thank you for purchasing loupes from us. Our series of adjustable loupes have been designed to meet the demanding needs of today's clinical professionals, which have the advantages of proper working distance, crystal clear vision and meticulous structure of the objects being observed. Taking advantage of our small, lightweight, clear vision and ergonomic loupes will certainly add ease to your day to day.

Maintenance and Care:

- Do Not Submerge your loupes in water, cleaning solutions or cold sterilization solutions: Although your loupes are water-resistant, we encourage cleaning them with a micro fiber cloth and antibacterial solutions sprayed on the cloth or directly on the frame and lens. Submerging the loupes in water can lead to a barrier seal being compromised and condensation forming in the barrels.
- Do Not autoclave or Chemiclave your loupes
- Do Not use cavi-wipes on your loupes as they are abrasive and remove the coatings we provide to offer proper protections from fogging, scratching and blue light protection.
- When using chemicals its best to dilute them 50% with water before applying to the loupes or a cloth for the loupes.

Cautions:

Your new loupes were thoroughly tested in both extreme heat and cold weathers to ensure the seal on the loupe barrels remains intact. Please avoid keeping your loupes in extreme temperatures exceeding 115 degrees Fahrenheit / 46 degrees Celsius or below -20 degrees Fahrenheit / -28.9 degrees Celsius.

Your new loupes are coated with a new blue light sensitive lens technology designed to reduce the amount of blue light to the eye by 75% or more. This new lens technology does not reduce the blue light capability by 100% so avoiding direct eye contact with curing lights is still suggested.

Warranty Coverage:

Dr.Brett'sPets offers the highest quality lifetime warranty in the industry. We back our products with a warranty that covers all manufacturer defects for life. Casual wear and tear and deliberate damage are not covered by this warranty and Dr.Brett'sPets reserves the right to cancel the warranty at any time.

Introduction to Your New Flip Up Loupes

A Few Key Points to know as you get familiar with your new Loupes.

- **Your loupes have a small turn dial in the middle of them seen with the blue arrow below. This Dial is the IPD (interpupillary distance) Adjustment. Simple put the loupes on and turn that dial until 2 circles converge to make one circle. This will allow your pupils to be directly behind the barrels allowing you to see 1 image.**



- Your loupes also have a hinge adjustment for allowing one to work with proper ergonomics. Understanding that many people who wear loupes will have different heights and ways to work allows for the user to adjust the hinges to make them easier to work both standing or sitting. The Hinge can be moved up, down and parallel for easy adjustments. The hinge is seen in figure 2 with the blue arrow. The hinge also allows the user to flip the loupe up when they are not using it.



Your new loupes come with a built in depth based on the magnification used. This depth allows the user to both work in a seated or a standing position. The magnification standard is arranged in a sense that the user can enhance their viewing capability to make any small object visible.